


Disposable 1870+ N95



1 Remove respirator from its packaging and hold with straps facing upward.

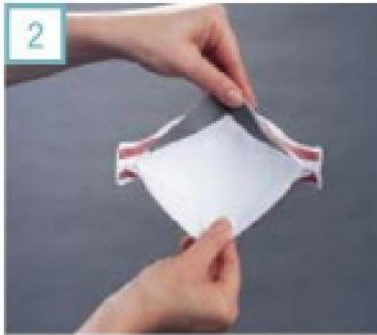

2 Fully open the top and bottom panels. Straps should separate when panels are opened. Make certain the bottom panel is unfolded and completely opened



3 Place the respirator on your face, so that the foam rests on your nose and the bottom panel is securely under your chin.



4 Pull the top strap over your head resting it high at the back of your head. Then, pull the bottom strap over your head and position it around your neck.



5 Adjust for comfort by pulling the top panel toward the bridge of your nose and position the bottom panel under your chin.



6 Place fingertips from BOTH hands at the top of your nose and hold the nosepiece to achieve a secure seal. **Do NOT pinch!**



Please Note:

Check the seal of your three-panel facepiece respirator each time you don the respirator.

Place one or both hands completely over the middle panel. Inhale and exhale sharply. If air leaks around your nose, readjust the nosepiece. If air leaks between the face and facepiece of the respirator, reposition it by adjusting the panels and straps. If you cannot achieve a proper seal, do not enter the contaminated area. See your supervisor.

REMOVAL INSTRUCTIONS



Without touching the respirator, slowly lift the bottom strap from around your neck up and over your head.



Lift off the strap on top of your head. Do NOT touch the respirator



Discard respirator after EACH use!

Reminder:

You must be fit-tested ANNUALLY to stay in compliance with UMMC's policy and for your SAFETY!

N95'S Limitations:

- N95's do NOT provide oxygen. Do NOT use it in an oxygen-deficient atmosphere.
- N95's do NOT protect against chemical challenges (cleaning solvents).
- N95's are a single-user item – do NOT share with others!

Any questions or concerns, please visit the **Fit Testing Center** located at N1E20 [Safety & Environmental Health](#) on the intranet
or
SafetyUMMC@umm.edu via email